

IVOCLAR VIVADENT ALL-CERAMICS

PREPARING FOR CEMENTATION – ZIRCONIUM OXIDE CERAMICS

	IPS e.max					
Material	IPS e.max ZirCAD / IPS e.max ZirPress			IPS e.max ZirCAD / IPS e.max Ceram		
	Zirconium oxide / Nano-fluorapatite glass-ceramic			Zirconium oxide / Nano-fluorapatite glass-ceramic		
Indication	Crowns and bridges w./w.o. pressed shoulder		Inlay-retained bridges	Crowns and bridges		
Cementation method	adhesive	self-adhesive* / conventional	adhesive	adhesive	self-adhesive* / conventional	
Blasting	Cleaning with Al ₂ O ₃ at max. 1 bar		—	Cleaning with Al ₂ O ₃ at max. 1 bar		
Etching	—		20 sec with IPS Ceramic Etching Gel	—		
Conditioning / Silanization	180 sec with Metal/Zirconia Primer	—	60 sec with Monobond-S	180 sec with Metal/Zirconia Primer	—	
Cementation system	Multilink® Automix	Vivaglass® CEM	Multilink® Automix	Multilink® Automix	Vivaglass® CEM	

* self-adhesive powder-liquid systems



Please read the corresponding Instructions for Use